


Disposable/Single-Use

# MaleUrinal

## FAQs

Product Code: **V20-015X**

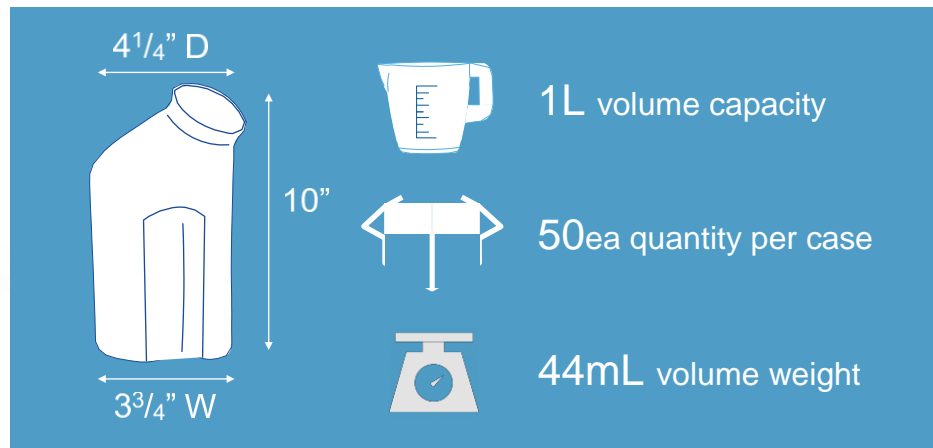

### Is it Leak-proof?

Yes, the Vernacare Pulp Paper Male Urinal is made of recycled newsprint mixed with a special resin that makes it leak-proof for a minimum of 4 hours. To extend the liquid retention time, add a 6gm Vernagel solidifier sachet (item# V36-001) to the male urinal, prior to use.

### What is its capacity?

1.5 Liters maximum capacity with a usable capacity of 1 Liter.

### Can it be reused?

The Vernacare Male Urinal is single-use only.

### Can it be recycled?

The Vernacare Male Urinal is biodegradable and can be recycled or deposited into landfill at the hospital's discretion. Always follow hospital protocol and red bag where required.

### How do I measure the contents?

For measuring liquid you may use a fluids measuring scale or pour the contents into a graduated vessel. The volume weight of an empty urinal is 44mL +/- 5ml.




For more Information:  
1(800) 523-8337 | [sales@macmed.com](mailto:sales@macmed.com) | [www.macmed.com](http://www.macmed.com)




# MaleUrinal

## Instructions for Use

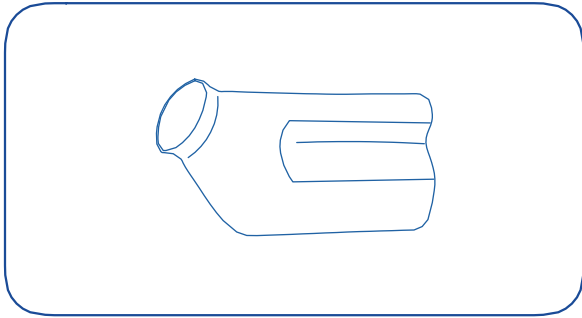


1

Place unused urinal on its side with the handle easily accessible to patient.

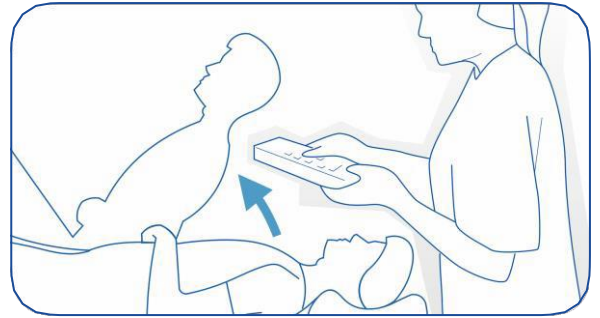


2

Adjust profiling bed to ensure patient is comfortable for toileting. Product performance will improve the more upright the patient is positioned.

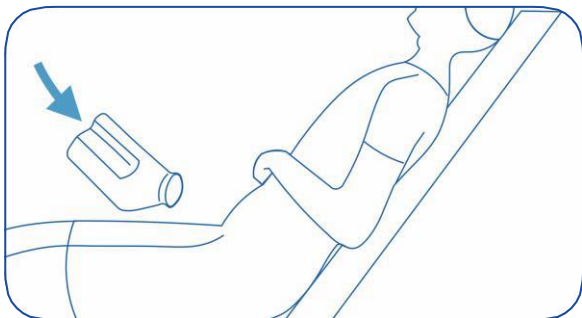


3

Position product, ensuring a comfortable fit to the patient's body. If the patient is able, they can position the product themselves.

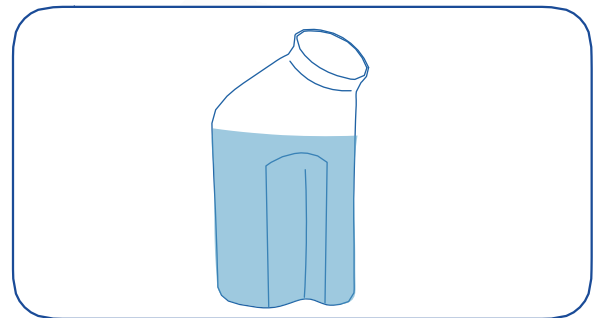

4

After use, instruct the patient to stand the urinal in the upright position, to indicate that the product has been used.

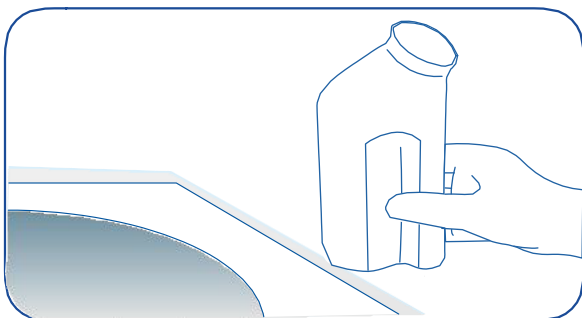

5

Carefully remove the product using the Handle.



6

Do **not** reuse. Replace with a new **Male Urinal** at the patient bedside